


DIARFLEX®

PRODUCT CATALOG

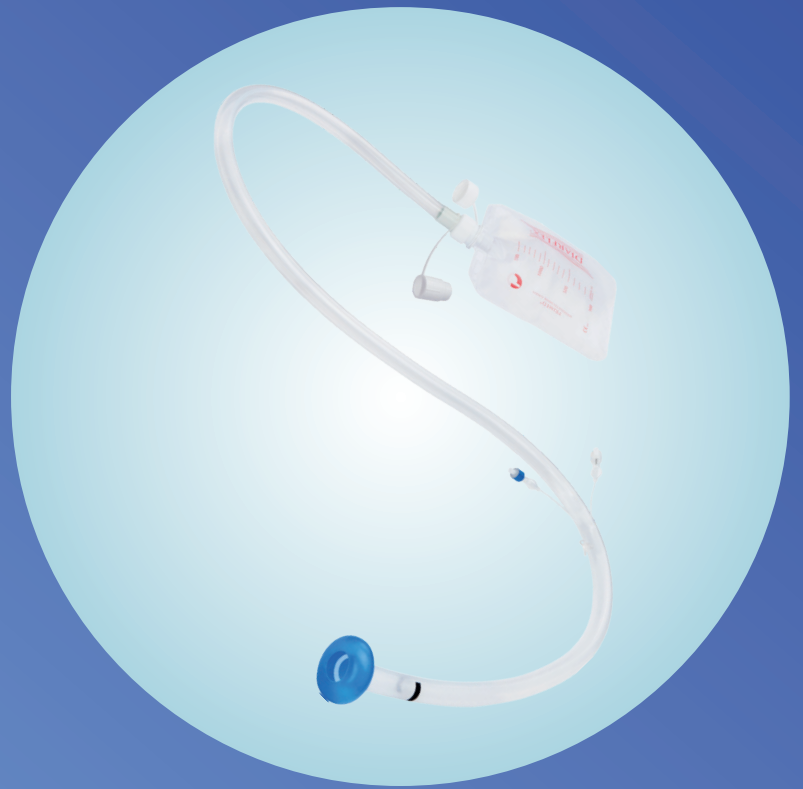

primed®



Primed[®] Medizintechnik GmbH is a highly experienced manufacturer of medical devices.

For many years, we have been supplying our customers with high quality products "MADE IN GERMANY".

Primed[®] both develops and manufactures all its products in-house employing a development process certified in accordance with DIN EN 13485.

From the initial product idea to material tests and tool construction right up to the product launch, we produce innovative medical devices from a single source. Our production spans the entire manufacturing process (injection moulding/ extrusion/ assembly) up to sterilisation.

These conditions enable us to react extremely flexible to market requirements and to ensure the continuous development of our products.

TABLE OF CONTENTS

DIARFLEX® Stool Management Systems.....	4
DIARFLEX®	4
DIARFLEX® Advanced	6
DIARFLEX® Faecal collector.....	9

The economic Stool Management System

PRACTICAL & EFFICIENT


- Hygienic discharging and collection of liquid, semi-liquid and infectious stools
- Reduces contact with infectious microorganisms both for the nursing staff administering treatment and the patient
- Reduces dermatological complications, because infectious stool can cause severe skin damage and even lead to decubitus
- Offers the possibility to flush the device if needed
- Reduces the development of unpleasant odours
- Guarantees dignified care of the patient by eliminating the need for diapers
- Enhances the efficiency of care provision (savings on cleaning costs and working time)
- Free from LATEX
- Free from DEHP

Catheter hose

- Made of soft, biocompatible Silicone



Rotary closure

- For securely attaching a collection bag

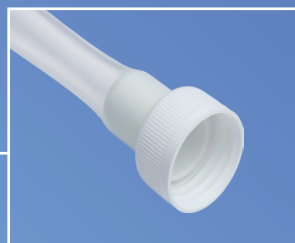

Low Pressure Balloon

- Assist in holding catheter securely in place in the rectal vault
- The soft, biocompatible material prevents irritation of the sphincter muscle

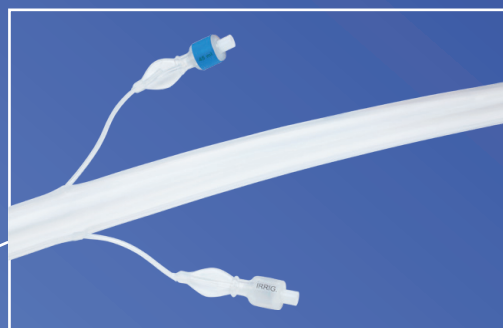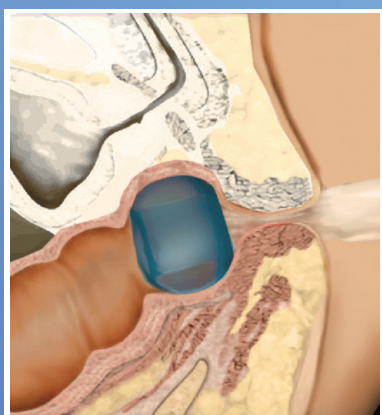

Connector for the Low Pressure Balloon (blue, „45 ml“)

- For filling the balloon with water or saline solution
- To empty the balloon


Irrigation connector (white, „Irrig.“)

- Used to flush the device if needed


The black marker provides better positioning control



- Permits easy insertion of the catheter in the unblocked state (therefore generously apply lubricant)

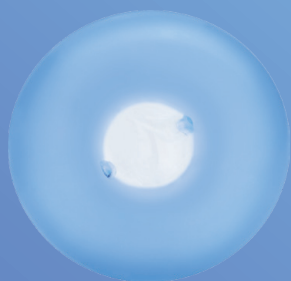

- In the blocked state, guarantees secure positioning of the catheter in the rectal opening
- Reduces the risk of leaks and liquid escaping

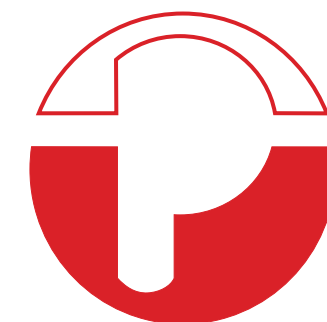

DIARFLEX® ADVANCED

The innovative Stool Management System

As refinement of our DIARFLEX® Stool Management System we developed DIARFLEX® ADVANCED.

BENEFITS:

- Offers the possibility to flush the device if needed and to take samples via the sampling port
- It is possible to rectally administer medication via the irrigation connector. For this purpose, the catheter clamp (included in delivery) enables closing the catheter system for a period of time determined by the physician
- Special odour-proof coating
- Includes hose plug for temporary seal of catheter hose during replacement of collection bag


Catheter clamp

- For closing the catheter system during rectal administration of medication
- included in the scope of delivery of DIARFLEX ADVANCED®

Low Pressure Balloon

- soft, biocompatible silicone balloon without recess between catheter hose and balloon

ADDITIONAL FEATURES

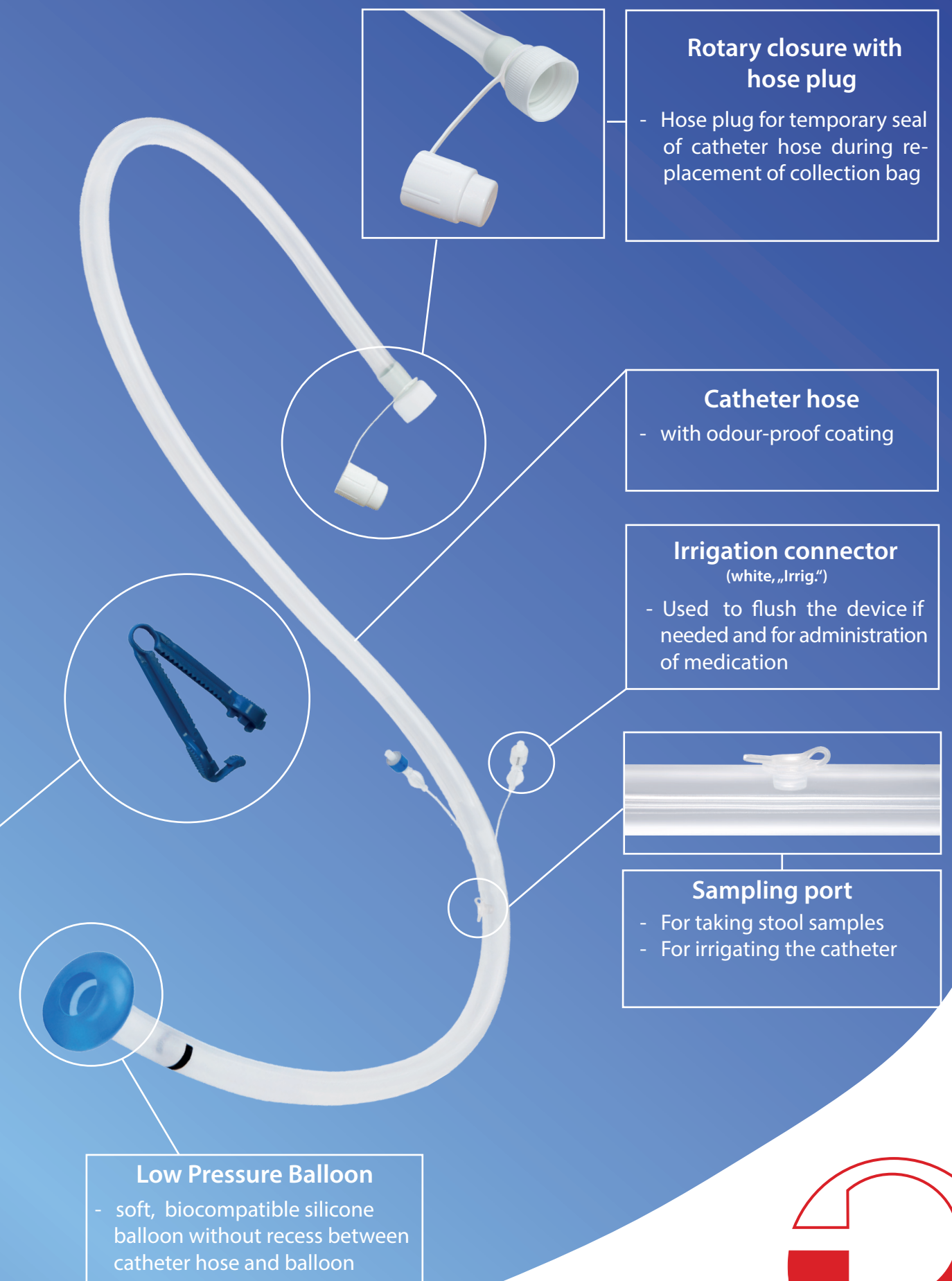



Collection bag for Diarflex® and Diarflex® Advanced

- To safely collect and dispose potentially infectious stools
- With superabsorbent polymer for the absorption of liquids
- Easy to connect to the hose end by means of a rotary closure
- With a firmly attached closure cap
- Capacity up to 1500 ml
- Available with an additional active carbon filter (included only in Diarflex® Advanced Stool Management Systems)

Replacement Collection Bags can be ordered separately

Additionally Included in the scope of delivery of both STOOL MANAGEMENT SYSTEMS:

- Two retaining facilities with velcro fastener for easy fixing of the system to the patient's bed
- Two replacement bags
- One syringe with Luer Lock connection for filling/ emptying the ballon and for flushing the system
- Warning labels for Fill lumen and Rinse lumen
- Instructions for use

Article number	Description	Packaging unit
20123	DIARFLEX® Stool Management System	1 set
20762	DIARFLEX® Advanced Stool Management System	1 set
20865	DIARFLEX® Replacement Bags with active carbon filter	2 pieces
20128	DIARFLEX® Replacement Bags	3 pieces

DIARFLEX® FAECAL COLLECTOR

The simple and effective draining system

For the hygienic channelling of liquid and semi-liquid stool and its secure collection for use with bedridden, immobilised or incontinent patients.



GENTLE TO THE SKIN: thanks to a hydrocolloid laster



INHIBITS ODOURS: using an active charcoal filter



COMFORTABLE: pouch has a resealable port with a cap



BENEFITS:

- Simple handling for enhanced care efficiency
- Inhibits unpleasant odours thanks to the integrated active charcoal filter
- Minimises contact with the stool
- Reduces dermatological complications (including decubitus)
- High wearing comfort thanks to hydrocolloid plaster that is gentle to the skin and absorbs moisture
- The especially thin form of the hydrocolloid plaster ensures **high flexibility** and **outstanding adhesive quality**

- **Resealable pouch port** for removal of liquid stool and for the connection of bed pouches during long-term use
- **Simplifies the calculation of fluid loss**
- **Collection volume 1.000 ml**
- **LATEX-free**
- **DEHP-free**

Article number	Description	Packaging unit
20775	DIARFLEX® Faecal collector	10 pieces

Non sterile product (usable up to 5 years)

MANUFACTURER

Primed[®] HALBERSTADT
 MEDIZINTECHNIK GMBH
 STRABE DES 20. JULI 1
 D-38820 HALBERSTADT
 PHONE: 0049 (0) 3941 - 668 6
 FAX: 0049 (0) 3941 - 245 65
 WWW.PRIMED-HALBERSTADT.DE
 PRIMED@PRIMED-HALBERSTADT.DE

DISTRIBUTOR